

Superior Thumb Spica

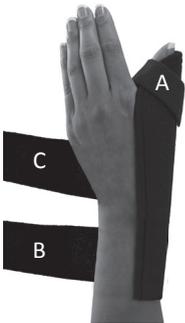


Figure 1



Figure 2



Figure 3

**NOT MADE WITH
natural rubber latex**

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

Intended for single patient use.



WARNING: If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

APPLICATION INSTRUCTIONS

1. Unfasten straps and slide affected thumb into brace. Wrap and secure thumb strap (A) ensuring a snug fit (Figure 1).
2. Pull and wrap forearm strap (B). Secure when desired fit and level of compression is reached (Figure 2).
3. Pull and wrap wrist strap (C). Secure when desired fit and level of compression is reached (Figure 2).
4. Adjust straps as needed when swelling of the hand, wrist or thumb changes (Figure 3).

NOTE: Once fitted, excessive strap length may be trimmed with scissors.

HEAT MOLDING INSTRUCTIONS

Heat molding should only be performed by medical professionals. Apply heat evenly to the area of interest using a heat gun. Heat only until soft enough to form. Brace should not be too hot to handle comfortably. Apply brace and press to mold into desired shape. Reheat and continue forming as necessary. Brace may also be trimmed with scissors.

NOTE: Please refer to sew-in label on device for care instructions.

Please visit our website at www.birdcronin.com for additional product information or contact a customer care representative at **800-328-1095**.

NOTICE: Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.

