

Sling and Swathe Immobilizer

NOTE: Instructions cover multiple product families, images may vary.



Figure 1



Figure 2



Figure 3



Figure 4

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

Intended for single patient use.



WARNING: If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

APPLICATION INSTRUCTIONS

1. Slide affected arm through the sling (A). Position sling (A) against rib cage and place each shoulder strap (B, C) directly over the left and right shoulders (Figure 1).
2. Wrap shoulder strap (B) diagonally across your back, under arm, and secure to sling close to elbow. Wrap shoulder strap (C) diagonally across your back, under arm, and secure to sling close to wrist (Figure 1). Straps will be crisscrossed against patient's back (Figure 2).

NOTE: Optionally, adjust the length of each strap by moving the metal buckle for a better fit.

3. Secure one end of the swathe (D) to the front of the sling (A), just above the arm (Figure 3).
4. Wrap the swathe (D) around the outside of the affected arm, along the rib cage, around the back, and under the unaffected arm (Figure 3).
5. Secure the swathe (D) to the front of the sling (A) (Figure 4).

NOTE: Please refer to sew-in label on device for care instructions.

Please visit our website at www.birdcronin.com for additional product and warranty information or contact a customer care representative at **800-328-1095**.

NOTE: This device is only one element in the overall treatment program administered by a medical professional. While every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort, there is no guarantee that injury will be prevented through the use of this product.

**NOT MADE WITH
natural rubber latex**