

Clavicle Brace

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.



Figure 1

Intended for single patient use.



Figure 2

NOT MADE WITH natural rubber latex



WARNING: If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

APPLICATION INSTRUCTIONS

1. Remove brace from package, loosen straps and lie product flat to identify left and right straps.
2. Drape left and right straps over appropriate shoulders and position felt pad between the shoulder blades.
3. Wrap left strap over left shoulder and under armpit. Feed strap through buckle opening, adjust and tighten by attaching hook and loop strap. Repeat process for right strap.
4. Adjust straps as need for comfort and posture.

NOTE: Please refer to sew-in label on device for care instructions.

Please visit our website at www.birdcronin.com for additional product information or contact a customer care representative at **800-328-1095**.

NOTICE: Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.

