

## Lumbosacral Support

**NOTE:** Instructions cover multiple product families, images may vary.



Figure 1

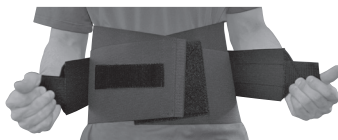


Figure 2



Figure 3

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

### Intended for single patient use.



**WARNING:** If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

### APPLICATION INSTRUCTIONS

1. Place support on the lower back and position for a comfortable fit. For supports with insert pocket, position panel on lumbosacral region of back with opening at the top.
2. Pull the left end across the abdomen and hold firmly against the right side (Figure 1).
3. Pull right end across on top and secure to left side of the abdomen.
4. Grip ends of side pulls to adjust as needed for the desired level of tension, support, compression and comfort (Figure 2).
5. For supports with an insert pocket, optional foam or moldable thermoplastic inserts are sold separately and can be added to provide extra support to lumbosacral region.

**NOTE:** Please refer to sew-in label on device for care instructions.

Please visit our website at [www.birdcronin.com](http://www.birdcronin.com) for additional product information or contact a customer care representative at **800-328-1095**.

**NOTICE:** Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.

**NOT MADE WITH  
natural rubber latex**

