

## Plantar Fasciitis Wrap



Figure 1

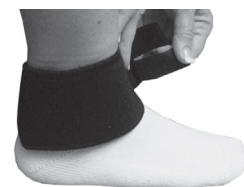


Figure 2



Figure 3



Figure 4

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

**Intended for single patient use.**



**WARNING:** If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

### APPLICATION INSTRUCTIONS

1. Remove and set aside the elastic strap from wrap.
2. Unfasten hook strips from wrap and place around the smallest part of the ankle, above Achilles. (Figure 1)
3. Secure hook to wrap. (Figure 2)
4. Apply the elastic strap by placing under the arch of the foot. Place strap in front of the heel.
5. Pull tabs up and back and attach to wrap. (Figure 3)
6. Adjust if necessary.

**NOTE:** Please refer to sew-in label on device for care instructions.

Please visit our website at [www.birdcronin.com](http://www.birdcronin.com) for additional product information or contact a customer care representative at **800-328-1095**.

**NOTE:** Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.

**NOT MADE WITH  
natural rubber latex**